



Bloomden A1-BL4 shade guide


Bloomden

INTRODUCTION

Dear Customer. Thank you for purchasing our Bloomden Shade Guide (A1 – BL4 shades) made of ceramic material. Bloomden A1-BL4 Shade Guide is used to determine the exact tooth shade. It is arranged in alphabetical order and divided into five groups (A, B, C, D, bleach), each group is further divided into different shades for the best approximation.

STRUCTURE

Shade series are divided into the following categories:

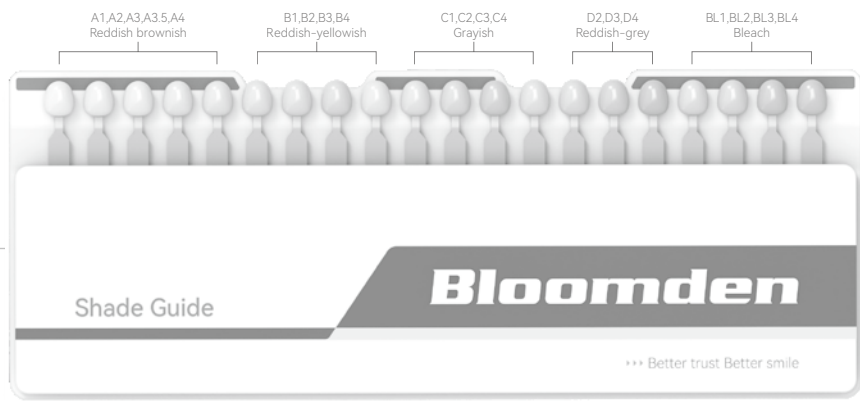

HOW TO USE

- 1) Hold the shade guide close to the patient's teeth to determine the closest match.
- 2) If the patient is female, ask them to remove any lip color or makeup to avoid contrast issues, which may affect shade determination. The surrounding colors should be as neutral as possible. Cover up bright clothing with a gray cape.
- 3) Patient must be sitting upright so that the doctor's eyes are at tooth level.
- 4) Try to determine tooth color under natural light and fluorescent lights, not under typical indoor lighting conditions.
- 5) Distance between the clinician's eyes and the patient's teeth should not exceed 15 inches.
- 6) Find the accurate shade by moving the guide in an edge-to-edge manner.
- 7) Make your selection quickly based on your first judgment, because the eyes will fatigue after about 5–7 seconds (retinal fatigue).
- 8) Disinfect the guide after each color measurement.

CARE INSTRUCTIONS

Bloomden A1-BL4 Shade Guide is made of ABS material, a high-quality, easy-to-maintain, high-temperature resistant (up to 150°C) material. The entire shade guide can be autoclaved at temperatures up to 140°C (sterilization as per gravity method: 121°C/250°F for 30 minutes, sterilization as per pre-vacuum method: 132°C/270°F for 4 minutes).

DISINFECTION

The disinfection process for this product is simple and easy. Mild spray disinfectants or wipes are the most recommended disinfection methods for shade guide. Most disinfectants are effective against HIV and HBV and are readily available according to the manufacturer's product information.

PRECAUTIONS

Bloomden A1-BL4 Shade Guide must be stored away from strong sunlight.

OTHER INFORMATION

Bloomden Bioceramics (HuNan) Co., Ltd
Address: No. 301, 5th Building, Hi-tech Int'l Enterprise Port, LianDong U Valley, Linyu Rd, Dongfanghong Street, Hi-tech Development Zone, Changsh, Hunan, China
Tel: (86) 731-8241-2982
Fax: +86 0731-82412982
E-mail: info@bloomden.com