

Bloomden Firing Paste

Instantly sticky without leaving any trace

Quick peeling No residue

Super strong support

• Better trust Better smile



Bloomden Firing Paste

Ceramic sintering support material

- Used to stabilize difficult-to-fix restorations during the sintering process in the porcelain furnace
- No discoloration of the restoration after firing
- Easy to peel off after sintering, no sandblasting and no residue
- No need to prolong the pre-drying time in the porcelain furnace

Fixable materials

Glass ceramics, zirconia, resin, metal, etc.


Main Ingredients	AL ₂ O ₃
	SiO ₂
	HEC
	PEG
	Distilled Water

How to use


1. Use after external dyeing of zirconia ceramic material



Squeeze out the Firing Paste and fill the crown until it is full

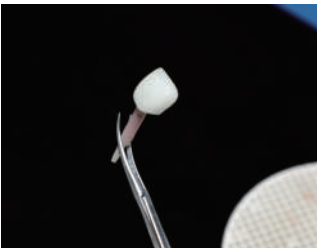

Insert the support pin into the crown and fix it

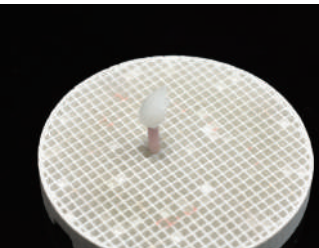

Insert the support pin into the sintering tray and fix it



Put the sintering tray into the porcelain furnace and start sintering



Take it out and clean it after sintering is completed

2. Use of glass ceramic materials (Same steps as above)

It does not affect the use of the restoration at all and retains its original color and perfect texture.